

Half-Face Reusable Respirators/Cartridge Infographic



User seal checks

Half-face reusable respirators | cartridge

NEGATIVE AND POSITIVE PRESSURE



Cover cartridge inlets with palms

Breathe **in** gently

Respirator should slightly **collapse**

If air leaks **in** around the face:



Adjust straps or
Reposition face piece



Cover exhalation valve with palm

Breathe **out** gently

Respirator should slightly **expand**

If air leaks **out** around the face:



REPEAT TEST UNTIL SEAL IS ACHIEVED

Source: safetyalliancebc.ca/covid-19